SUBJECT INFORMATION

**What You Should Know About the
OraSure® HIV-1 Oral Specimen
Collection Device Prior to Providing
an Oral Specimen**

**Followed by
Information on
HIV/AIDS and
the OraSure HIV-1
Device**

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SUBJECT INFORMATION

WHAT YOU SHOULD KNOW ABOUT THE ORASURE HIV-1 ORAL SPECIMEN COLLECTION DEVICE

Testing an OraSure HIV-1 specimen for HIV-1 antibodies is highly accurate; however, testing a blood specimen is more accurate. You have a choice.

- Studies show that for every 100 people with HIV antibodies in their blood, tests using OraSure HIV-1 oral fluid specimens may miss 1 or 2 people.
- Studies show that for every 100 people with HIV antibodies in their blood, tests using OraSure HIV-1 oral fluid specimens may give an indeterminate test result to 2 or 3 people instead of a positive test result.
- Before using OraSure, read the HIV and AIDS information in this pamphlet. This will help you to decide if you are at risk of being infected with the HIV virus. It also explains steps you can take to protect yourself and others from becoming infected with the HIV virus.

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Negative Results

- A negative result usually means no HIV infection, but it cannot rule it out completely.
- If you have risk factors described in "How does someone get HIV?" section of this pamphlet, you should consider being retested at a later date.

Positive Results

- A positive result means that you may have an HIV infection.
- If you test positive for the first time, the Association of State and Territorial Public Health Laboratory Directors (ASTPHLD) recommends that HIV testing be repeated using a new sample for all persons who test positive for the first time*. The repeat testing can be done on either a second OraSure or a blood specimen.

Indeterminate Results

- An indeterminate result cannot be interpreted as a positive or as a negative test result; therefore, it is not known whether you are, or are not, infected with HIV.
- HIV testing should be repeated using a new sample.

* Report and Recommendations, ASTPHLD Fifth Consensus Conference on Testing for Human Retroviruses. March 1990.

If you decide to use OraSure HIV-1, follow the instructions of the person trained to collect the oral specimen.

* NOTE: Full descriptive information about the OraSure HIV-1 Oral Specimen Collection Device is available from the trained collector or from the ordering physician.

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HIV & AIDS

What are HIV and AIDS?

HIV is the virus that causes AIDS. It is the **Human Immunodeficiency Virus**. Usually a person has the virus for months or years before any signs of illness appear. It slowly weakens the body's ability to fight off illness. People with AIDS can have serious infections and cancers. These illnesses make them very sick and can eventually kill them.

How does someone get HIV?

HIV spreads through blood, semen, vaginal fluids or breast milk from infected people to uninfected people. People get HIV from contact with these fluids. Contact can come from unsafe sex. It can also come from sharing used needles and syringes. Infected women can pass the virus to their babies during pregnancy, childbirth and breast feeding. Some people who received blood products from 1978 to 1985 received infected blood. Now blood banks test all blood for HIV before they use it.

People do not become infected with HIV through everyday casual contact with people at school, work, home or anywhere else. The virus is not spread from contact with sweat, tears, saliva, or a casual kiss from an infected person (deep, or "French" kissing is not advised). Nor can it be contracted from contact with forks, cups, clothes, phones, toilet seats or other things used by someone who is infected with HIV. People do not become infected from eating food made by an HIV-infected person. And to date, people have not become infected with HIV through bug bites.

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How can I avoid becoming infected?

The best way to avoid getting HIV is to avoid activities that allow the virus to be passed to you. By using the following suggestions, you will lower your risk of getting HIV:

- The only way to avoid sexual exposure to HIV is to have sex in a monogamous relationship with an uninfected partner, or to abstain.
- If you are not in such a relationship, and engage in sex, you should use a latex condom correctly every time you have sex.
- Do not share needles, syringes or injection equipment.

Why should I get tested?

You cannot always tell by looking at someone whether he or she has an HIV infection. The virus may take time to show its effects. A person can have HIV for ten years or more before the symptoms of AIDS appear. Anyone can have the virus without knowing it. The only way to be sure, for yourself and your loved ones, is to take the test.

It is important to find out if you have HIV so that you do not give it to someone else. If you know you have HIV, you can avoid any activity that may pass it on.

It is important to find out if you have HIV so that you can receive good medical care. There are medicines that can help keep you healthy even though you carry the HIV virus.

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What should I know before I test?

Your health care provider is the best person to answer your questions about HIV, the OraSure HIV-1 antibody test, and other testing options.

You have a choice of the type of test to use. The presence of HIV antibody testing can be performed on either an OraSure specimen or a blood specimen. Ask your health care provider for the information you need to make good choices.

- What is the OraSure HIV antibody test and how it is done?
- What are HIV and AIDS?
- How does HIV spread and how can people prevent it?
- How private is my test result?
- What results are possible from my test? What do the results mean?
- How could the test results affect me?
- Who should I tell about my test results?

When you are tested for HIV, a blood or OraSure HIV-1 specimen will be collected. Your specimen will be sent to a testing laboratory. The laboratory will check your specimen for HIV antibodies. The presence of HIV antibodies in your body means that you have been infected with the AIDS virus.

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What does a positive result mean?

A positive result usually means you have an HIV infection. If you test positive for the first time, it is recommended that you repeat HIV testing using a new sample (Association of State and Territorial Public Health Laboratory reference). Early detection means that you may benefit from special medical care. New treatments can help keep you healthy, even though you have HIV. See a doctor, even if you don't feel sick. A doctor can help you stay healthy and live longer. Other tests can tell you how strong your immune system is and what treatments might be best for you. Some people stay healthy for a long time with HIV. Others may become ill more rapidly. Be careful not to pass HIV on to others.

What does a negative result mean?

A negative result means the lab found no HIV antibodies in your specimen. If you had no risks for HIV in the six months before your test, you probably were not infected with HIV. A risk is having contact with blood, semen, or vaginal fluids from a person who may be infected with HIV. Read "How can I avoid becoming infected?" Talk to your health care provider about your result.

If you had any risks of becoming infected with HIV in the six months before your test, it is still possible that you have HIV. This is because your body can take six months after you are infected to make HIV antibodies. Also, sometimes test results are wrong. Results are wrong more often with OraSure HIV-1 specimens than with blood specimens. Get tested again in three to six months to be sure you are not infected. You can stay free of HIV by following prevention guidelines. Practice safe sex. Ask your health care provider to help you understand what your result means for you.

What does an indeterminate result mean?

An indeterminate test result cannot be interpreted as a positive or a negative test result; therefore, you may or you may not be infected with HIV. To find out if you are infected or not infected, you should repeat HIV testing using a new sample as soon as possible.

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Where can I get more Information?

If you have any questions, ask your health care provider. You can also call the National AIDS Hotline at 1-800-342-AIDS (1-800-342-2437). HIV specialists staff the Hotline. They can give you quick, private answers at any time, day or night. Your local health department is another place to go for information. An AIDS service organization near you can be a good source for information, education and help.

What is OraSure HIV-1?

The OraSure HIV-1 Oral Specimen Collection Device is a device used to collect oral fluid from the mouth in only a few minutes without using needles. The oral fluid is tested to see if it contains HIV antibodies.

If you decide to have an OraSure HIV-1 sample tested for HIV antibodies, a trained collector will ask you to place the OraSure HIV-1 Oral Specimen Collection Pad between your lower cheek and gum and gently rub the pad back and forth until it is wet, then leave it in place for two minutes. The collector will ask you to put the pad in a vial and snap off the stick. Your sample will be sent to a laboratory for HIV-1 antibody testing.

OraSure HIV-1 specimen collection is painless and involves no needles. There will be no visible sign that you have been tested. Having an OraSure HIV-1 sample tested for HIV antibodies is a very accurate method. Blood testing, however, is more accurate. Be aware that you have a choice of having either a blood specimen or an OraSure HIV-1 sample taken for HIV testing.

REFERENCE

Novello, M.D., M.P.H., Antonia Coello. Surgeon General's Report to the American Public on AIDS.

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Manufactured by:

OraSure Technologies, Inc.

220 East First Street
Bethlehem, PA 18015
Made in USA
1-800-869-3538

Authorized Representative:

OraSure Technologies, Inc.
Fokkerstraat 11
2811 EN Reeuwijk
The Netherlands

www.orasure.com

© 2001, 2003 OraSure Technologies, Inc.

OraSure® is a registered trademark of OraSure Technologies, Inc.

Item# 3001-0866 (rev. 10/03)